# Fecal Calprotectin Sample Collection Kit

*KAPEPKT843*

---

Version: 230816

Date of issue : 16/08/2023

Revision date: 16/08/2023

# History

---

**Summary of change :**

<b>Previous Version :</b>	<b>Current Version :</b>
230313	230816 SPECIMEN COLLECTION Clarification of step 4, 5 and 6.

## Fecal Calprotectin Sample Collection and Extraction

An Advanced & Clean Method for Processing Fecal Sample for the Aid of Calprotectin Measurement

# KAPEPKT843

## IN VITRO DIAGNOSTIC

DIAsource ImmunoAssays SA - Rue du Bosquet 2, B-1348 Louvain-la-Neuve, Belgium - Tel: +32 10 84 99 11 - Fax : +32 10 84 99 91

### INTENDED USE

Calprotectin Sample Collection Kit is a device intended for human fecal sample collection and extraction. The extracted sample can be used directly for calprotectin level determination by professional laboratories. For in-vitro diagnostics purpose only

### REAGENTS AND MATERIALS PROVIDED

1. Fecal Specimen Collection Device: 50 units/kit. Each tube contains 4.5mL Fecal Calprotectin Sample Collection and Extraction buffer.
2. Instructions for Use

### SAFETY PRECAUTIONS

The reagents must be used in professional laboratory environment and are for in vitro diagnostic use only. Wear gloves while performing sample collection and treat the reagent as if they are hazardous. The buffer may cause irritation on skin upon contact. Do not get in the eyes, on skin, or clothing. Never transfer any reagents with mouth. Do not ingest or inhale. On contact, flush with copious amounts of water for at least 15 minutes. Exercise Good Laboratory Practices.

### MATERIALS REQUIRED BUT NOT PROVIDED

1. Plastic wide-mouth stool collection container


### SPECIMEN COLLECTION

1. Open the lid of primary stool collection container
2. Open the device sampling lid (orange color) and pull the sampling lid out of the tube.
3. Use the grooved region of the wand to stir and mix soft stool or dip into hard stool in multiple times (3 - 5 sites) to collect a small amount of stool sample.
4. Insert the sampling wand and screw the sampling lid back into the sampling tube in a vertical position. Do not spill any solution from the tube. Tightly seal the sampling lid against the tube. Let the sample soak in extraction buffer at least 10 min.
5. Vortex the tube on upright position at least 10 seconds to dissolve the stool sample completely.
6. The extracted sample should be on upright position on a rack for 1 – 5 minutes to sediment stool particles before assay.

### PROCEDURAL NOTES

1. This device is recommended for the secondary collection of stool sample in professional laboratories, while patient should perform primary collection of stool sample into a plastic wide-mouth stool collection container in bathroom.
2. Careful technique is necessary to ensure reproducibility and maximum detective sensitivity.
3. For liquid stool, please use pipet to add 100 µL sample directly into Fecal Calprotectin Sample Collection Tube.

### SAMPLE COLLECTION PROCEDURE

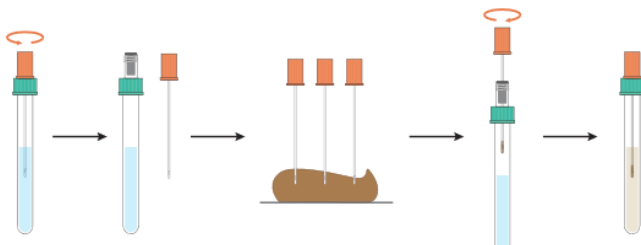### STORAGE

The extracted stool sample should be tested within 8 hours, or stored at 2-8 °C and tested within 3 days. It may be stored below - 20 °C for a longer storage period. Avoid more than three freeze - thaw cycles for each specimen.

### WARRANTY

This product is warranted to perform as described in its labeling and literature when used in accordance with all instructions. DIAsource ImmunoAssays DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, and in no event shall DIAsource ImmunoAssays be liable for consequential damages. Replacement of the product or refund of the purchase price is the exclusive remedy for the purchaser. This warranty gives you specific legal rights and you may have other rights, which vary from state to state.

### TECHNICAL ASSISTANCE AND CUSTOMER SERVICE

For technical assistance please contact [products.support@diasource.be](mailto:products.support@diasource.be)  
To place an order, please contact [customer.service@diasource.be](mailto:customer.service@diasource.be)

Other translations of this Instructions for Use can be downloaded from our website: <https://www.diasource-diagnostics.com/>

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